



# Installing ABPM Software

**Step 1.** Visit [www.cardiostudy.ca](http://www.cardiostudy.ca)


**Step 2.** In the top right corner of the website, click on Downloads

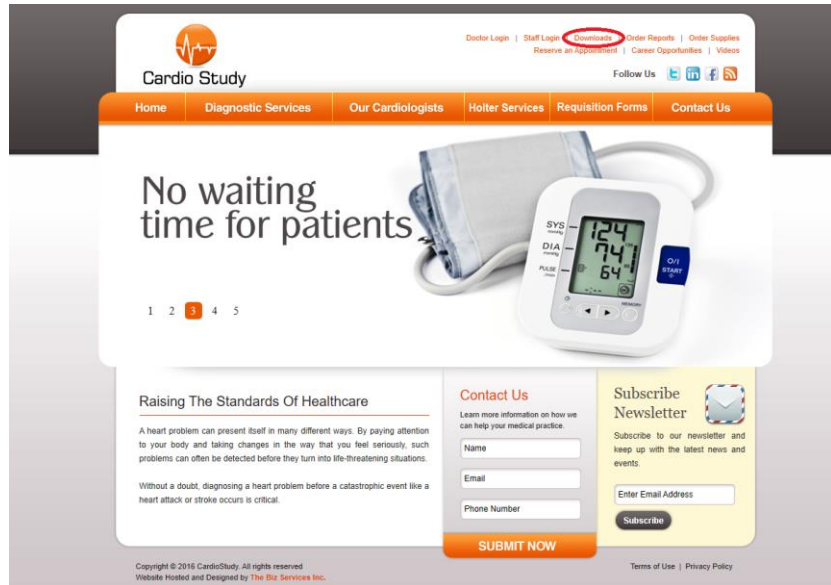



**Step 3.** On the next page, click to download "ABPM Software"

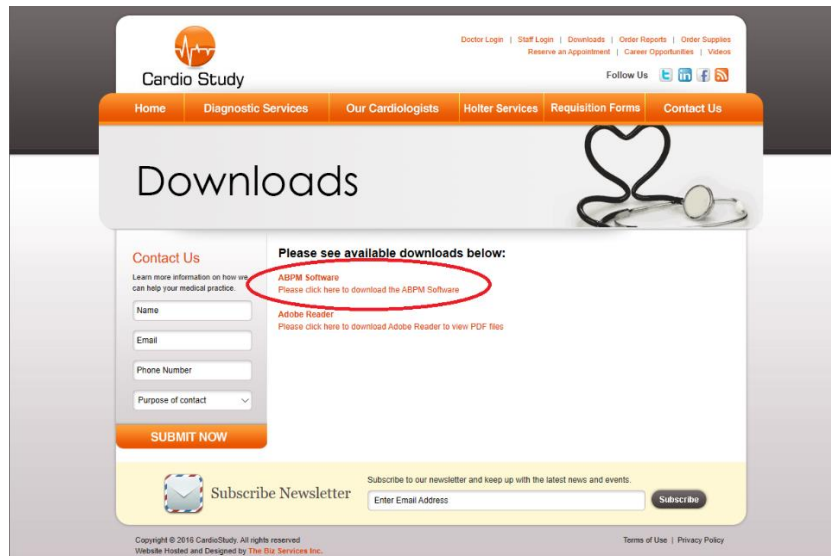


**Step 4.** If prompted to "Run or Save" the file, click on Run. **Otherwise** click "Save" and open the file once it is downloaded.

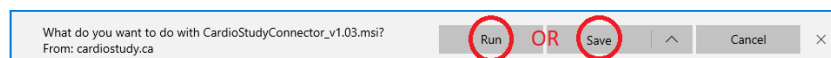


**Step 5.** During installation, you may be prompted by your computer to allow the program to install. In this event, click on "More Info", followed by "Run Anyway"

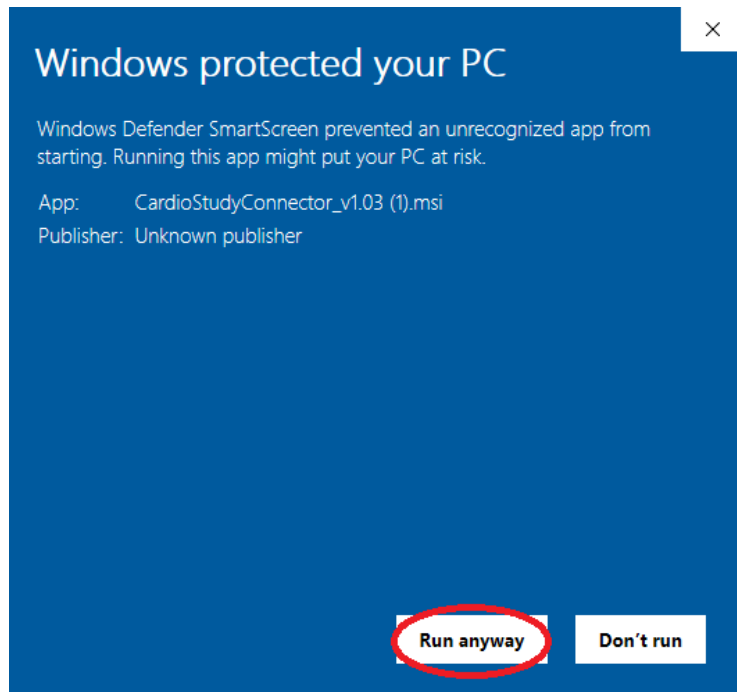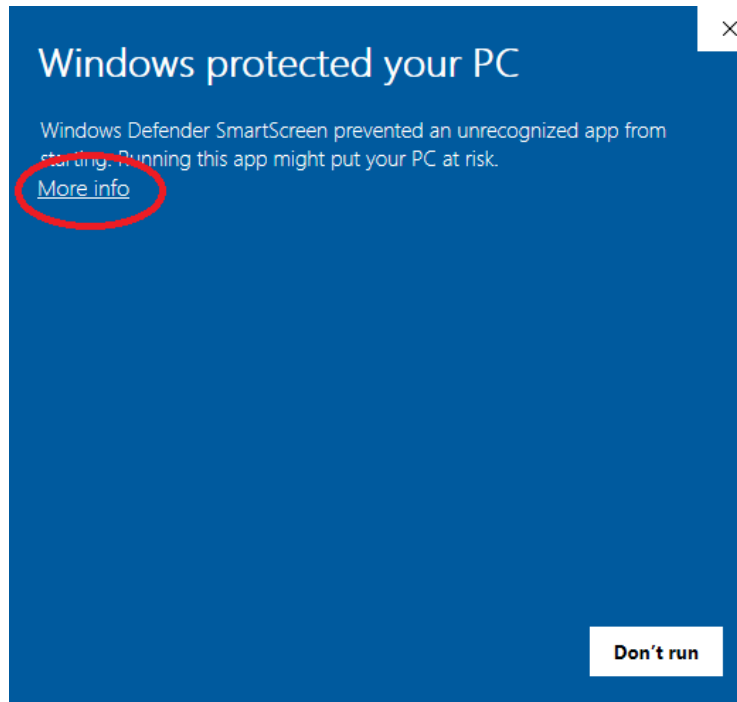


**Step 6.** Next you will see the installation wizard pop up. Click on "Next"

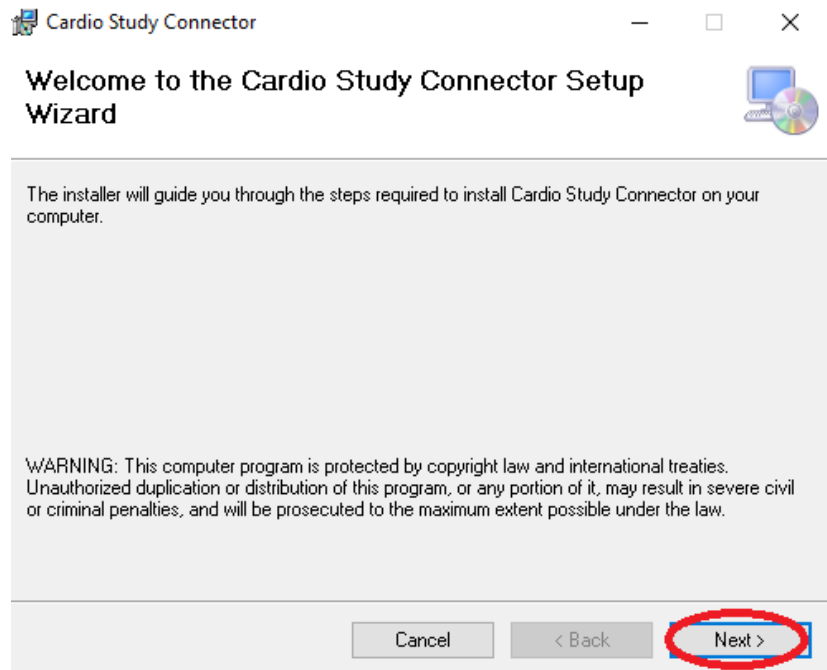


**Step 7.** On the next screen, select "Everyone", then click "Next".

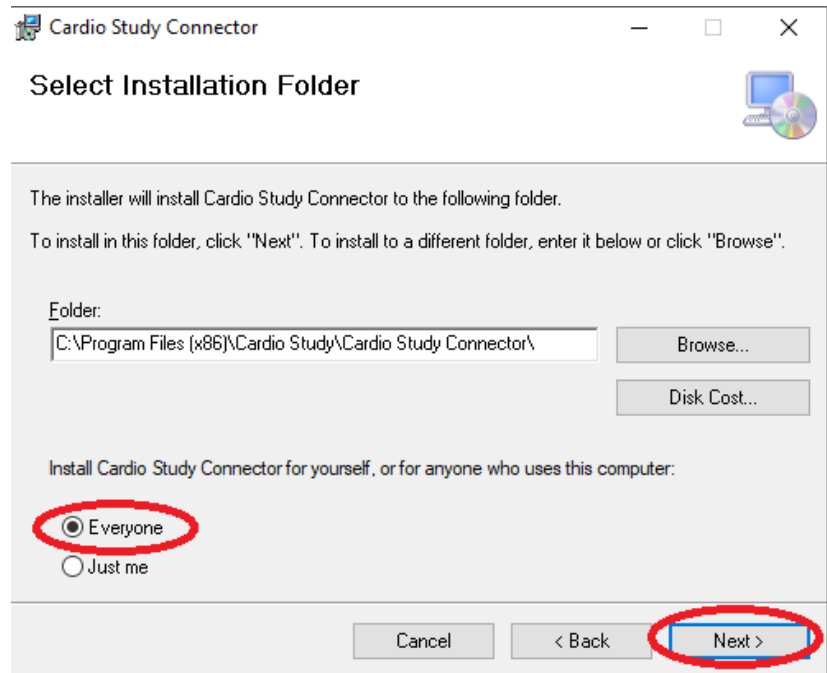

**Step 8.** On the next screen, click “Next”.

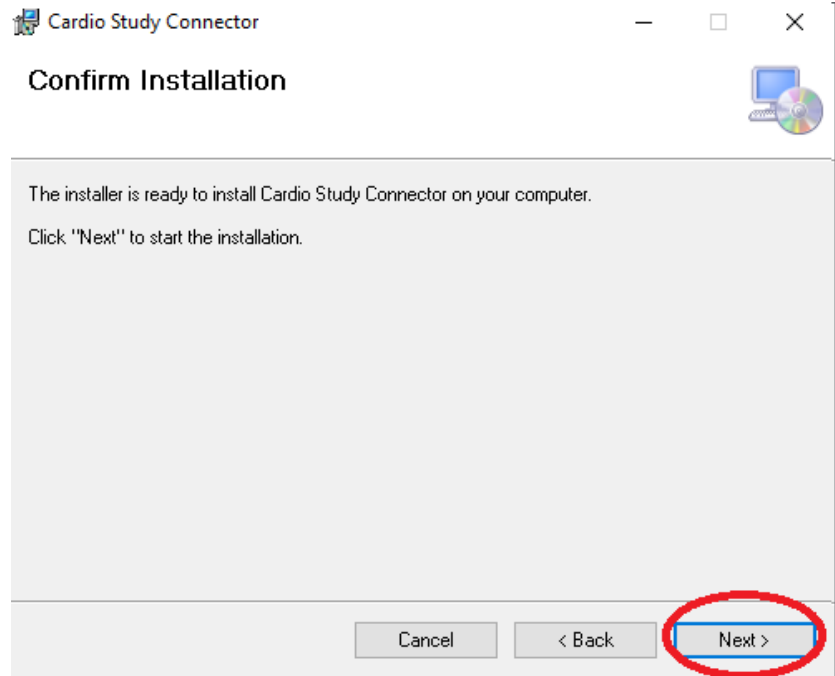

**Step 9.** You may be prompted by your PC to grant permission to install the file. Click “Yes”.

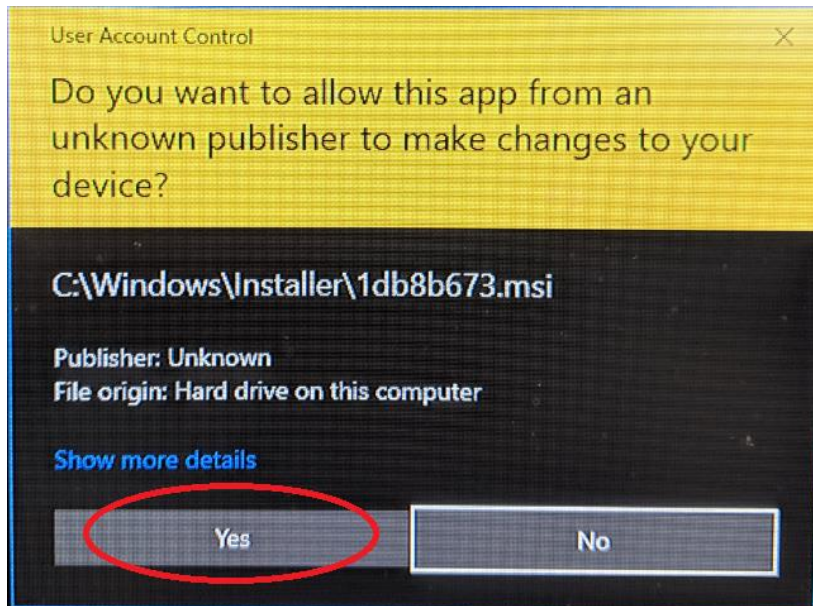

**Step 10.** Once the installation is complete, you may click "Close".

