


# ABPM Tutorial – Retrieving Results

(Please note – these instructions should be followed after the patient testing has been completed.)

**Step 1.** The ABPM test must be stopped first. This can be done by removing the batteries from the ABPM.

**Step 2.** Connect the ABPM to your PC using the provided USB cable

**Step 3.** A small window will pop up. Enter your credentials and click on “Test Completed”

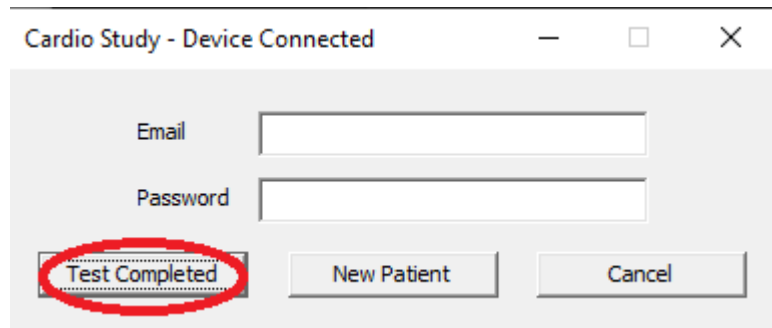


Cardio Study - Device Connected

Email

Password

**Test Completed** New Patient Cancel

**Step 4.** You will be prompted with a window saying “Download Completed”. Click “OK”. A new window will automatically open in your web browser.

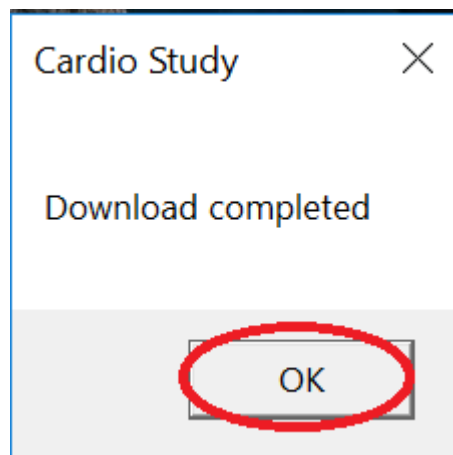

Cardio Study

Download completed

**OK**

**Step 5.** Complete the report by entering the remaining fields.

The screenshot shows a web interface with a top navigation bar containing four tabs: 'Statistical' (circled in red), 'Interpretation', 'Graphical', and 'Raw Data'. Below the navigation bar is a 'Patient Information' section with several input fields. Red arrows point to the following fields: 'Health Card (or Blue Cross, IFH, etc.)' (value: 1234567890), 'Referring Doctor (full name)' (value: Dr. John Doe), 'Indication for Test' (value: High BP), 'Read By' (value: In-House), 'First Name' (value: Bob), 'Last Name' (value: Jones), 'Birthdate' (value: 1990/04/02), 'Gender' (value: Male), 'Clinic Address' (value: 123 Clinic Road, Toronto, ON), 'Height (cm)' (value: 182), 'Weight (kg)' (value: 99), and 'Medications' (value: None). A green arrow points to the 'Read By' dropdown menu. At the bottom right, there is an orange button labeled 'Assign' with a checkmark icon.

Fields that need to be filled out include:

- Referring Doctor Name
- Indication For Test
- Read By
- Patients Birth Date
- Gender
- Clinic Address
- Height (cm)
- Weight (kg)
- Medications

**Please note – “Read By”**


You may choose to have Cardio Study physicians finalize the report, or your physicians may finalize the report in-house.

**Step 7.** Once all information is filled out, click “Assign”

This is a close-up view of the bottom portion of the form. It shows the 'Clinic Address' field (value: 123 Clinic Road, Toronto, ON), 'Height (cm)' field (value: 182), and 'Weight (kg)' field (value: 99). Below these is the 'Medications' field (value: None). At the bottom right, the orange 'Assign' button is circled in red.



**Step 10.** You can always pull up previous reports by visiting the Cardio Study Portal. To view patient reports online visit [portal.cardiostudy.ca](https://portal.cardiostudy.ca) and log in using your credentials. Once logged in, click on “ABPM” on the left side of your screen to access your patient list.

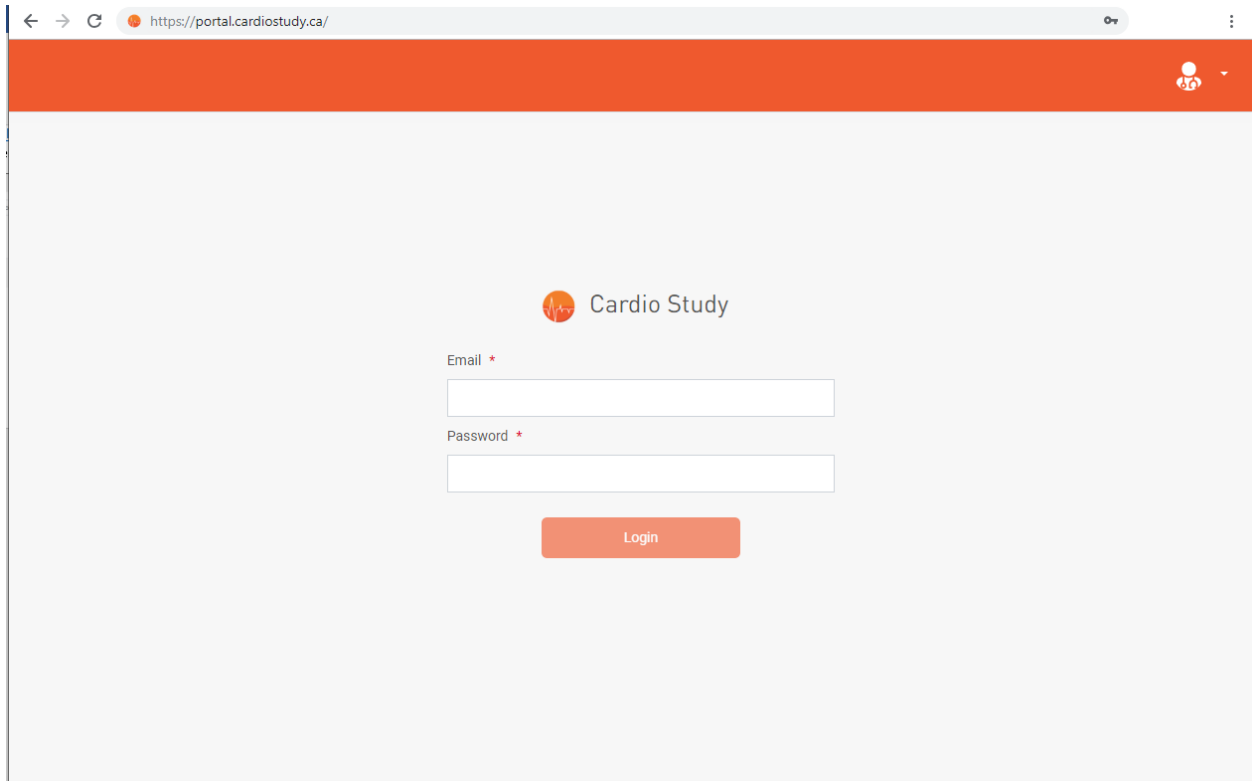

The screenshot shows a web browser window with the address bar displaying <https://portal.cardiostudy.ca/>. The page features a prominent orange header bar at the top right containing a user profile icon and a dropdown arrow. The main content area is light gray and centered, displaying the "Cardio Study" logo (a heart with a pulse line) and the text "Cardio Study". Below the logo are two input fields: "Email \*" and "Password \*", each with a red asterisk indicating a required field. A red "Login" button is positioned below the password field.